


# Sling and Swathe Immobilizer


## APPLICATION INSTRUCTIONS

CP020404 Rev A 04/08


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

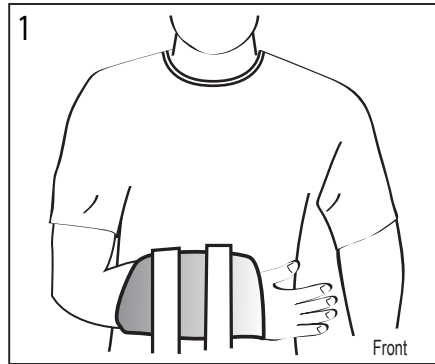


1. Position the Sling  
Place the affected arm into the sling.

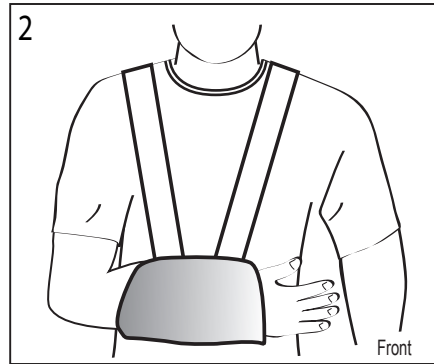


2. Position the Straps  
Take the straps over the shoulders.

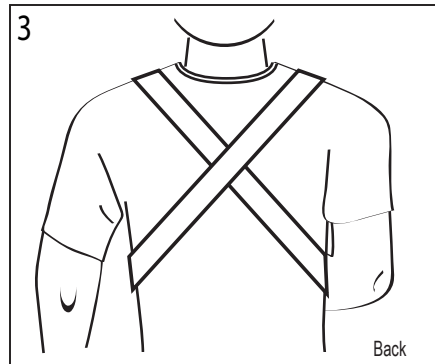


3. Cross Straps in Back

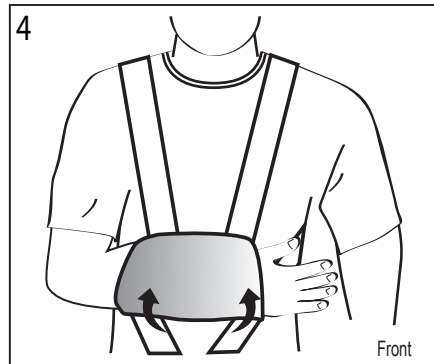

4. Secure Straps  
Bring straps under the sling and attach to the front of the sling.

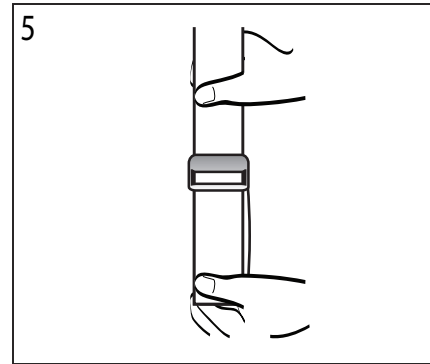

5. Adjust the Straps  
Pull straps to desired length and tension by pulling on the strap end.

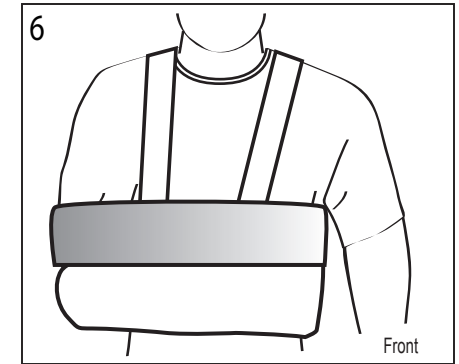

6. Attach the Swathe  
Wrap the swathe around the chest and injured arm above the sling. Secure using the hook and pile fasteners to the swathe.

**INDICATIONS:** Indicated for immobilization of the upper extremity.

**CONTRAINDICATIONS:** Contraindicated for unstable fractures of the upper extremity.

**WARNINGS:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1.888.BLEDSOE (253.3763).



Council Directive 93/42/EEC  
of 14 June 1993 concerning Medical devices

MANUFACTURED BY:  
**BLEDSOE BRACE SYSTEMS™**  
2601 Pinewood; Grand Prairie, Texas, 75051, USA  
Toll Free Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666  
Local: 972.647.0884 • Local Fax: 972.660.5495  
International Fax: 972.606.0649  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE

EC

REP

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: (+31) 70 345 8570  
Fax: (+31) 70 346 7299