

# Fracture Bracing Plates

## Application Instructions CP020106 Rev. D 3/01



Council Directive 93/42/EEC  
of 14 June 1993 concerning  
Medical devices



The winged tibial plate (shown left) is pressed into position before hinge-arms are applied to the brace. The regular tibial plate can be pressed into position before or after hinge-arms are added.



The femoral plate can be pressed into position before or after hinge-arms are added.



The hinge arms are applied over the condyle control wings of the winged tibial fracture plate and pressed to engage hook to pile closure.




Hinge arms do not contact the femoral plate or regular tibial plate.



Fasten straps per the Bledsoe Brace instructions. The photo at left shows the position of plates to hinge area.



Brace shown with regular tibial plate and shoe insert.

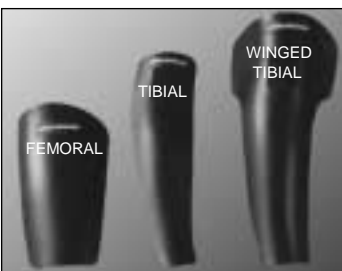

### Sizing & Ordering Information

Brace Size	Femoral		Tibial		Winged Tibial		
	Left	Right	Left	Right	Left	Right	
Small	22"-24"	AJ005101	AJ005201	AL005101	AL005201	AL004101	AL004201
Medium	26"-28"	AJ005105	AJ005205	AL005105	AL005205	AL004105	AL004205
Large	30"-34"	AJ005109	AJ005209	AL005109	AL005209	AL004109	AL004209

This accessory is offered for sale by or on the order of a physician or other qualified medical professional. This accessory is not intended for reuse on a second patient or for extended use beyond 180 days. This accessory is generally not intended for patients in excess of 250 lbs. (114 kilos). This accessory is not intended to prevent injuries or reduce or eliminate the risk of reinjury. **WARNING:** Do not leave this accessory in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this accessory, seek medical attention.** **Warranty:** This accessory is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to accessories that have not been modified or subjected to misuse, abuse or neglect.

Manufactured by:  
**Bledsoe Brace Systems™** by Medical Technology Inc. 2601 Pinewood, Grand Prairie, Texas, 75051, USA  
Toll Free Tel. 1-888-BLEDSOE (253-3763) Or 1-800-527-3666 • Local Tel. 972-647-0884  
Local FAX 972-660-5495 International FAX 972-606-0649 • [www.bledsoebrace.com](http://www.bledsoebrace.com)

**For product information or questions pertaining to sales or service, please contact the national distributor in your area or the manufacturer.**

European Authorized Representative:  
**MEDPASS International Limited**  
Windsor House  
Barnett Way  
Barnwood  
Gloucester GL4 3RT  
United Kingdom  
+44(0)1 452-619-222