

Bledsoe LC Boot (Preassembled)

Application Instructions CP020169 Rev. A 12/05



Council Directive 93/42/EEC
of 14 June 1993 concerning
Medical devices

LIGHTWEIGHT WALKER FOR SHORT TERM APPLICATION

From the family whose name means braces



This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Indicated for tarsal, metatarsal, forefoot, and calcaneus fractures, forefoot sprains, and grades 1, 2, & 3 ankle sprains. Also indicated for fractures of the distal tibia or fibula.

Contraindications: Contraindicated for unstable fractures and proximal tibial or fibular fractures.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.**

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.bledsoebrace.com or by calling Bledsoe Brace Systems.

Manufactured by:

Bledsoe Brace Systems™ by Medical Technology Inc.
2601 Pinewood, Grand Prairie, Texas, 75051, USA
Toll Free Tel. **1-888-BLEDSOE** (253-3763)
Or 1-800-527-3666 • Local Tel. 972-647-0884
Local FAX 972-660-5495
International FAX 972-606-0649
www.bledsoebrace.com

European Authorized Representative:

MEDPASS International Limited
Windsor House
Barnett Way
Barnwood
Gloucester GL4 3RT
United Kingdom
+44(0)1 452-619-222

For product information or questions pertaining to sales or service, please contact the sales representative in your area or the manufacturer.

Bledsoe LC Boot (Preassembled) Application Instructions



1

1. Apply a clean, dry sock on the foot.
Warning: Never wear the boot without a sock.



2

2. Remove the strap ends from the D-rings and open the cuff closure. Position the leg in the boot with the leg in line with the uprights. If necessary, trim the excess cuff, leaving 2"-3" (5cm-8cm) of overlap.



3

3. Close the hook-to-pile closure at the front.



4

4. Pass the forefoot strap through the D-ring. Tighten the strap and engage to the hook piece.



5

5. After engaging the forefoot strap to the hook piece, trim off the excess strap if necessary .



6

6. Starting at the ankle, fasten the strap through the D-ring at the front of the leg and tighten the strap. Repeat this step for the remaining two straps.



7

7. Remove the plastic sleeve and engage the hook-to-pile closures on the upright with the wrap.