


Bledsoe Wee Walker Boot

Application Instructions CP020133 Rev H 02/07

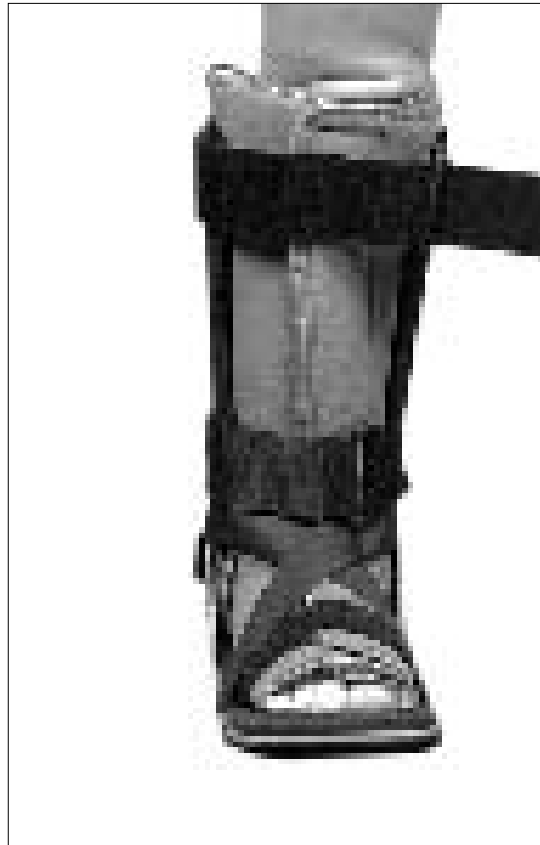

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs. (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Indicated for post-injury or post-operative application to grade 1, 2, or 3 ankle sprains, and stable or internally fixed fractures of the foot or ankle. Also indicated for treatment of stable or internally fixed fractures or fractures with bridging callous of the middle to distal tibia or fibula.

Contraindications: Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. If any additional pain or symptoms occur while using this device, seek medical attention.

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.bledsoebrace.com or by calling (800) 527-3666.

Manufactured by:
Bledsoe Brace Systems™
2601 Pinewood, Grand Prairie, Texas, 75051, USA
Toll Free Tel. 1-888-BLED SOE (253-3763)
Or 1-800-527-3666 • Local Tel. 972-647-0884
Local FAX 972-660-5495
International FAX 972-606-0649
www.bledsoebrace.com

European Authorized Representative:




Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299

ORTHOMEDICO (European Bledsoe Agent)
+32 54 504005 www.orthomedico.be

For product information or questions pertaining to sales or service, please contact your area sales representative or Bledsoe Brace Systems directly.

Bledsoe Wee Walker Boot

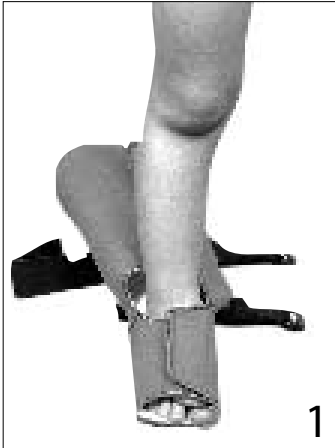

1

1. APPLY CALF WRAP

Slip the foot into the bootie with the toes under the toe piece. Close the forefoot cover over the foot. Trim the excess leaving 2"-3" (5cm-7.6cm) overlap.



2

2. Wrap the calf section around the leg, and trim the excess leaving 2"-3" (5cm-7.6cm) overlap. Close the hook to pile fastener at the front. Remove the forefoot strap from the uprights. Pull center of strap to loosen.

Note: Do not pull the strap out of the slots on the side of the boot.



3

3. SLIP FOOT INTO BOOT

Slip the foot into the boot with the plastic covers over the uprights to prevent the hook to pile engagement.



4


4. Position the uprights in line with the leg and remove the plastic covers to engage the hook to pile fastener.



5

5. FASTEN STRAPS

Pull the ends of the forefoot strap to tighten. Cross each end of the strap over the forefoot and attach to the hook to pile fastener on the opposite uprights.

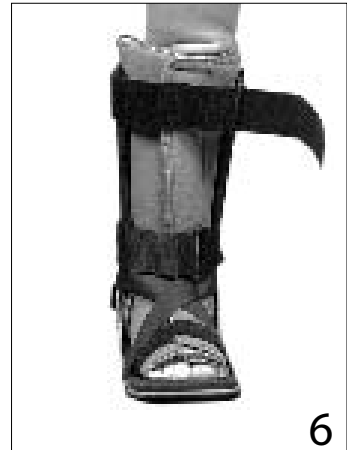

6

6. Starting at the ankle, wrap the strap ends securely forward around the sides of the leg to interlock the strap with the uprights

Note: Bottom strap overlaps part of the forefoot strap to help secure the forefoot strap in place. Repeat this step for remaining strap.