



**Bledsoe  
Brace  
Systems™**

## Bledsoe M•Brace (Air & Air•Gel)



Cool and open design for complete and lightweight comfort

Easy to apply, adjusts to fit most patients

- **Get back in the game:** Reduces ankle swelling while boosting patient's confidence when its time to get back in the game.
- **Availability:** The Bledsoe M•Brace has two models to choose from; the Air M•Brace or the Air•Gel M•Brace, and are conveniently one size and a universal style.
- **Lightweight and comfortable:** Cool and open design provides complete and lightweight comfort.
- **Easy application:** Easy to apply adjusting to fit most patients, and can be adjusted without the aid of tools.
- **Reduce swelling:** The Air•Gel Brace is available for reduction of swelling, and can be used for cold therapy.

When your ankle demands support for minor sprains



# Bledsoe M•Brace (Air & Air•Gel)

## Restrictions on use

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs. (114 kg). This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

## Warning

Do not leave this device in the trunk of a car on a hot day or in any place where the temperatures may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.



**Bledsoe  
Brace  
Systems™**

by Medical Technology, Inc.

2601 Pinewood Drive  
Grand Prairie, TX 75051 USA

1-888-BLEDSOE  
(1-888-253-3763)

1-800-527-3666

972-647-0884

Local Fax:  
972-660-5495

International Fax:  
972-606-0649

[www.bledsoebrace.com](http://www.bledsoebrace.com)

## Indications

Indicated for immediate application to ankles for mild sprains.

## Contraindications

Contraindicated for grade two and three sprains where plantar or dorsi flexion can not be achieved.

## Ordering Information

### M•Brace (Air & Air•Gel)

	Air M•Brace	Air•Gel M•Brace	Air / Air•Gel M•Brace
Single	ES040005	ES050005	N/A
20 Pack	ES042005	ES052005	ES062005 (Half Quantity of Each)
50 Pack	ES045005	ES055005	ES065005 (Half Quantity of Each)

#### Cold Therapy

- Remove therapeutic gel bag from ankle support system
- Place in freezer for 2 hours
- Place gel bag back in ankle support and follow fitting instructions
- Absorbent sock should be worn before ankle support is fitted
- DO NOT place entire ankle support into freezer

#### Precautions

- Patients should always consult their physician or therapist for proper therapy instructions prior to using the therapeutic ankle support system
- Always wear an absorbent sock before cold therapy is applied
- The entire ankle support system can be hand washed with mild soap and water. Do not apply artificial heat to dry
- Only freeze the therapeutic bag. **DO NOT** place the entire system in freezer

## Bledsoe limited warranty

This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, pads, straps and normal wear components. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect.



For more detailed information on this brace, including fitting and adjustment instructions, visit our Web site:  
[www.bledsoebrace.com](http://www.bledsoebrace.com)

**For product information or questions pertaining to sales or service, please contact the sales representative in your area or Bledsoe Brace Systems.**

