



**Bledsoe
Brace
Systems™**

Bledsoe Sport & Sport Max Braces



Sport Front Closure Brace

**Easy application
and adjustable
tension**



Sport Max Brace

- **The Bledsoe Sport and Sport Max Braces** combine the use of a dual offset hinge with straps and a durable neoprene. The surface contact area of the neoprene provides warmth to the injured area as well as biofeedback to enhance proprioception and joint position sense. The Sport Max Braces provide added support through the three-point leverage of the straps and hinge.
- **Sport and Sport Max Front Closure:** Wrap around front closure style allows easier application and adjustable tension.
- **Sport and Sport Max:** Durable neoprene sleeve and pull tabs for easy "pull on" application and comfort.
- **Support:** Patients with minor knee ligament strains and stretches or meniscal damage can benefit from added support to help prevent more serious injury. These four braces are designed to match the right kind of support for these very mild instabilities.
- **Options:** Optional patellar buttress for stabilizing patella.

Bledsoe Sport & Sport Max Braces

Restrictions on use

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

Warning

Do not leave this device in the trunk of a car on a hot day or in any place where the temperatures may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.



**Bledsoe
Brace
Systems™**

by Medical Technology, Inc.

2601 Pinewood Drive
Grand Prairie, TX 75051 USA

1-888-BLEDSOE
(1-888-253-3763)

1-800-527-3666
972-647-0884

Local Fax:
972-660-5495

International Fax:
972-606-0649

www.bledsoebrace.com

Indications

Indicated for mild sprains of the medial or lateral collateral ligaments, mild injuries of the menisci, or mild injuries to the patellar retinaculum, to provide increased support, compression and warmth, and aid in proprioception.

Patellar Buttress: Indicated to help control patellar tracking and aid proprioception.

Contraindications

Contraindicated for injured or reconstructed knees that have moderate to severe instability.

Ordering Information

Sport & Sport Max Front Closure Brace

HCFA Code: L1820

	Thigh Circumference	Brace Size	Sport Front Closure		Sport Max Front Closure	
			without patellar buttress	with patellar buttress	without patellar buttress	with patellar buttress
	(38 - 44.5 cm) 15" - 17.5"	XS	RK070001	RK071001	RK080001	RK081001
	(44.5 - 51 cm) 17.5" - 20"	S	RK070003	RK071003	RK080003	RK081003
	(51 - 57 cm) 20" - 22.5"	M	RK070005	RK071005	RK080005	RK081005
	(57 - 63.5 cm) 22.5" - 25"	L	RK070007	RK071007	RK080007	RK081007
	(63.5 - 69.5 cm) 25" - 27.5"	XL	RK070009	RK071009	RK080009	RK081009

To determine size, measure thigh circumference 6" (15 cm) above the knee.

Sport & Sport Max Brace

Thigh Circumference	Brace Size	Sport		Sport Max	
		without patellar buttress	with patellar buttress	without patellar buttress	with patellar buttress
(38 - 44.5 cm) 15" - 17.5"	XS	RK010001	RK011001	RK020001	RK021001
(44.5 - 51 cm) 17.5" - 20"	S	RK010003	RK011003	RK020003	RK021003
(51 - 57 cm) 20" - 22.5"	M	RK010005	RK011005	RK020005	RK021005
(57 - 63.5 cm) 22.5" - 25"	L	RK010007	RK011007	RK020007	RK021007
(63.5 - 69.5 cm) 25" - 27.5"	XL	RK010009	RK011009	RK020009	RK021009

For **custom sizes** contact Customer Care for additional information.

Bledsoe limited warranty

This device is warranted to be free from defects in material and workmanship for a period of 90 days for wraps, straps and normal wear components, 12 months on all other parts. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect.



For more detailed information on this brace, including fitting and adjustment instructions, visit our Web site:
www.bledsoebrace.com

For product information or questions pertaining to sales or service, please contact the sales representative in your area or Bledsoe Brace Systems.



CP010169 Rev. E 2/04